DE Calcium hydroxide LC (GB) Light-curing calcium hydroxide Instructions for Use

DE Calcium hydroxide LC is a light-curing radiopaque one-component calcium hydroxide material for linings and temporary long-term sealing. The material can also be applied directly and cured with halogen light. Its high consistency avoids resorption and dissolution symptoms.

Indication:

DE Calcium hydroxide LC is indicated for deep cavities to reduce the pulp reaction on sensitivity (isolation layer). In direct proximity of the pulp, the application of an aqueous calcium hydroxide preparation is recommended.

The light-curing radiopaque one-component calcium hydroxide bonds to materials, e.g. crown and bridge materials, temporary sealants on methacrylate basis, certain impression materials (e.g. polyether).

Application:

1. The cavity should be prepared according to the standard rules of the dentistry.
2. The areas where the dentin is covering the pulp have to be dried thoroughly and any contamination should be avoided.
3. Place the required quantity of the DE Calcium hydroxide LC onto a mixing pad and apply the material immediately with a suitable instrument. Avoid direct light and close the syringe properly. In case of linings of more than 1 mm, apply and polymerize layer by layer.
4. Cure the material with halogen light. Keep the light source as close as possible to the surface (approx. 1-2 mm) and cure for at least 40-60 seconds. In case of more distance, extend curing time.
5. Carefully remove excess material with a rotating instrument and apply the filling according to the manufacturer's instructions.

Precautions

To prevent marginal leakage and infiltration of bacteria in the cervical /approximal margins, a permanent isolation is recommended (e.g. a light-curing one-component bonding **DE Bond 97-90324**) to cover the baseliner.

Do not remove excess material with a probe, this could eventually remove the whole lining (use a rotating instrument). DE Calcium hydroxide LC contains methacrylates and amine. Do not apply the material in case of patients with known allergies against these ingredients.

Eugenol and other contaminations (e.g. thymol) prevent the polymerisation, therefore avoid contact with these materials.

Too short curing times and insufficient light intensity lead to incomplete curing. For this reason, check the halogen lamp/light guide regularly.

Storage:

Store the material at room temperature. Reclose the syringe carefully after use. Avoid humidity.

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