INTRODUCTION:

DE Healthcare Micro-Hybrid Composite is a light cured resin-based composite which contain approximately 79% by weight inorganic filler with an average particle size of 0.6 microns. All shades but Incisal are radiopaque. It is recommended that the composite is stored at ambient temperature. Shelf life of the composite is 3 years from manufacture date. Expiration date is marked on unidose refill package. Do not use after expiration date.

<input type="checkbox"/>	97 92238	A1 enamel	<input type="checkbox"/>	97 92244	B3 enamel
<input type="checkbox"/>	97 92239	A2 enamel	<input type="checkbox"/>	97 92245	C1 enamel
<input type="checkbox"/>	97 92240	A3 enamel	<input type="checkbox"/>	97 92246	C2 enamel
<input type="checkbox"/>	97 92241	A3.5 enamel	<input type="checkbox"/>	97 92247	C3 enamel
<input type="checkbox"/>	97 92242	A4 enamel	<input type="checkbox"/>	97 92248	D3 enamel
<input type="checkbox"/>	97 92243	B2 enamel	<input type="checkbox"/>	97 92249	universal opaque

INSTRUCTION FOR USE

Working area preparation:

1. The use of a rubber dam is recommended to assure dry working area.
2. Clean the tooth surface using a rubber cup and abrasive paste free from fluorides and oil. Then rinse it thoroughly with water spray and dry it with air jet (not contaminated by oil).
3. Gently prepare the cavity with minimum reduction of dental tissues.
4. Treat (the prepared and clean) cavity walls, enamel and dentine using the technique of total etching with orthophosphoric acid 40% (DE Etch Gel) in the form of gel. Rinse the cavity with water spray and dry it. Etching time: enamel 30 sec, dentine 15 sec. Rinse the etching gel with water spray for 30 sec and gently dry with clean air jet.
5. Warning: Avoid contact of the etching gel with the oral cavity mucus or with skin as well as accidental contact with eyes. In case of contact with surrounding tissues (mucosa or skin) rinse with plenty of water and seek special medical attention, if necessary.
6. Apply adhesive

system on etched and dried dental tissues of the prepared cavity walls in accordance with the manufacturer's directions.

Placement:

1. Select the desired shade of the composite.
2. When using capsule delivery, insert Tip and properly align within Dispenser.
3. Place the Tip at the deepest portion of the preparation. Caution: Extrude slowly with even pressure.
4. Increments should be no more than 2mm at a time. After placing an increment, stroke the composite to ensure marginal adaptation.
5. Light Cure each increment for a minimum of 40 seconds (Recommended cure times: see manufacturer's recommendation). When multiple surfaces are available, cure each surface for this recommended time.

SHADES:

The following is a listing of the shades available:

Tips 20 x 0.2g: A1, A2, A3, A3.5, A4, B2, B3, C1, C2, C3, D3, Universal Opaque

ATTENTION:

Tips are designed for single patient use only. Do not re-cap and/or re-use the tip once material has been dispensed for that patient.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

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