# Safe R Evac System liquid

IMPROVED  
FORMULA

97-92533

## Evacuation System Disinfection concentrate

- High performance solution designed for cleaning and disinfecting of dental suction system and mouth rinse basin.
- Solución de alto rendimiento diseñada para la limpieza y desinfección de sistemas dentales de aspiración y palanganas de enjuague bucal.
- Concentré de haute performance conçue pour le nettoyage et la désinfection du système d'aspiration dentaire et du crachoir.
- Zur Reinigung und Desinfektion von dentalen Absauganlagen und Mundspülbecken.
- Soluzione dalle alte prestazioni per la pulizia e la disinfazione di impianti di aspirazione odontoiatrici e lavabi risciacquo bocca.
- Concentraat voor het reinigen en desinfecteren van een tandheelkundig afzuigsysteem en het spittoon.

**Indications for use:** Safe R Evac System liquid must be first be diluted with water – to a 2 or 3 % concentration. (20/30 ml concentrate with 1 ltr. water). Approx. 1 liter of solution should be sucked through. At normal usage the evacuation unit should be disinfected one to two times daily. Do not mix with other disinfectants or cleanser. The solution should not be left in the suction unit overnight. **Microbiological Effectiveness:** Bactericide in accordance to VAH/DGHM EN 13727, EN 14561 and EN 1276. Levurocide (Candida albicans) in accordance to VAH/DGHM and EN 13624, EN 14562, EN 1650. "Restricted viruses" (effective against Membrane covered viruses e.g. HBV,HIV, HCV) in accordance to BGA (RKI)/DVV. **Effectivity:** Bacteria and Candida albicans in accordance to VAH/DGHM as Instrument disinfection (high bio load): 3,0% 5min. 2,0% 15min. Bacteria in accordance to EN 1276(high bio load): 2,0% 5min. Candida albicans in accordance to EN 1650 (high bio load): 2,0% 5min. Bacteria in accordance to EN 13727 and EN 14561(high bio load): 3,0% 2min. Candida albicans in accordance to EN 13624 and EN 14562 (high bio load): 3,0% 2 min. Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA(RKI)/DVV and EN 14476 (bio load): 2,0% 2min. **Safety instruction:** R 36/38 Irritating to eyes and skin. S 24/25 Avoid contact with skin and eyes. S 26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S 28 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S 37/39 Wear suitable gloves and eye/face protection.

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Contents:  
Contenido/Contenuto:  
Inhalt/Contenuto:  
Inhoud:

5L e  
alcohol-free

**DEHP**™ **DE Healthcare Products**  
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribuit per/Vertriebt durch/Distribuito da:  
DE Healthcare Products, Gillingham ME8 0SB U.K., [www.dehpsbrand.com](http://www.dehpsbrand.com)

# Safe R Evac System liquid

IMPROVED  
FORMULA

## Evacuation System Disinfection concentrate

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