Impression Disinfection

- **Mould disinfectant concentrate for the disinfection and cleaning of dental moulding materials.**
- **Desinfectante concentrado para desinfectar y limpiar materiales de moldeado dental.**
- **Concentré pour désinfecter et nettoyer les matériaux destinés aux empreintes dentaires.**
- **Abformdesinfektionsmittelkonzentrat zur Desinfektion und Reinigung dentaler Abformmaterialien.**
- **Disinfettante concentrato per modelli per disinfettare e pulire i materiali dentali di stampo.**
- **Afdrukdesinfectiemiddelconcentraat voor de ontsmetting en reiniging van dentale afdrukmaterialen.**

Instructions for usage: The mould disinfectant is a disinfectant concentrate that needs to be diluted with water prior to usage. Place moulds in the usage solution. All surfaces to be disinfected must be completely wetted by the usage solution. Remove the moulds following the prescribed disinfection period and thoroughly rinse with water. Use the solution again depending upon the degree of contamination. Do not mix with detergents or other disinfectants. **Microbiological Effectiveness:** Bactericidal in accordance to VAH/DGHM; EN 13727, EN 14561; levurocical (*Candida albicans*) in accordance to EN 13624, EN 14562. Tuberculocidal in accordance to VAH/DGHM. "limited virucidal" (effective against membrane covered viruses, i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV. **Effectivity:** Bacteria in accordance to VAH/DGHM EN 13727, EN 14561 (high bio load); 2,0% 5 min, *Candida albicans* in accordance to VAH/DGHM, EN 13624, EN 14562 (high bio load); 2,0% 5 min, Tbb (Tuberculosis) in accordance to VAH/DGHM (high bio load); 4,0% 15 min, Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA (RKI)/DVV (BVDV/Vaccinia according to EN 14476); 2,0% 5 min. **Safety instruction:** R 22 Harmful if swallowed. R 34 Causes burns. S 26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S 28 After contact with skin, wash immediately with plenty of . (to be specified by the manufacturer). S 37 Wear suitable gloves. S 45 In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). S 60 This material and its container must be disposed of as hazardous waste.

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Contents:
Contento/Contenuto:
Inhalt/Contenuto:
Inhoud:

1Le
alcohol-free

DEHP™ **DE Healthcare Products**
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribuido por/Versand durch/Distribuido de:
DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehpbroad.com

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