Safe R Instrument lemon concentrate

IMPROVED
FORMULA

97-92536

Instrument Disinfection

- For cleaning and disinfecting of rotating and surgical instruments.
- Limpieza y desinfección de instrumentos giratorios y quirúrgicos de todo tipo.
- Destiné au nettoyage et à la désinfection des instruments chirurgicaux et rotatifs de tous types.
- Zur Reinigung und Desinfektion von rotierenden & chirurgischen Instrumenten aller Art.
- Per la pulizia e la disinfazione di strumenti rotatori e chirurgici di tutti i tipi.
- Voor het reinigen en desinfecteren van een verscheidenheid aan roterende en chirurgische instrumenten.

Indications for use: Safe R Instrument Lemon concentrate form which has to be diluted with water. Heavy contaminated instruments should first be cleaned before submerging into the disinfectant. After disinfectant, instruments are to be thoroughly rinsed with water and dried. Do not mix with other types of disinfectant or cleansers. **Microbiological Effectiveness:** Bactericide, levurocidal (Candida albicans), tuberculocida, "restricted viruses" (effective against membrane covered viruses, i.e. HBV, HIV, HCV). **Effectivity:** Bacteria in accordance to VAH/DGHM, EN 13727, EN 14561 (high bio load): 3% 15min. 2% 30min 1% 60min. Levurocidal in accordance to VAH/DGHM and EN 13624, EN 14562 (high bio load): 3% 15min. 2% 30min 1% 60min. Ttb (Tuberculosis-inactivation in accordance to VAH/DGHM): 3% 60min. Membrane covered viruses (i.e. HBV, HIV, HCV): 3% 15min. 1% 60min. Ultrasonic bath (Bacteria and Candida albicans in accordance to VAH/DGHM: (high bio load): 3% 5min. **Safety Precautions:** R 34 Causes burns. R 42 May cause sensitisation by inhalation. S 23 Do not breathe aerosols. S 26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S 37/39 Wear suitable gloves and eye protection. S 45 In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). S 60 This material and its container must be disposed of as hazardous waste. Contains piperazine, Bis(3-aminopropyl)dodecyl amine

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Contents:
Contenido/Contenuto:
Inhalt/Contenuto:
Inhoud:

5L e
alcohol-free

DEHP™ **DE Healthcare Products**
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribué par/Vertreibe durch/Distribuito da:
DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehpsbrand.com

Safe R Instrument lemon concentrate

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 **DE Healthcare Products**
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribue par/Vertrieb durch/Distribuito da:

DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehprand.com