## Safe R Non-alcoholic disinfection Wipes refill

97-92571

### Disinfection Wipes alcohol free

- Alcohol free wet wipes for rapid surface disinfection of non-invasive medical device.
- Toallitas húmedas sin alcohol para la desinfección rápida de productos no invasivos sanitarios.
- Tissu imbibé sans alcool, pour la désinfection rapide des dispositifs médicaux non invasifs.
- Alkoholfreie Feuchttücher zur schnellen Desinfektion von nichtinvasiven Medizinprodukten.
- Fazzolettini umidi privi di alcol per una rapida disinfezione dei dispositivi medici non invasivi.
- Alcoholvrije vochtige doekjes voor snelwerkende desinfectie van niet-invasieve medische hulpmiddelen.

**Indications for use:** Surfaces should be thoroughly wiped with the disinfectant tissue and leave to rest. For bigger surfaces of sensitive material such as artificial leather, try it on a smaller scale (area) first to test compatibility. To avoid streaks from appearing a regular wiping with a wet (water) disposable cloth or tissue is recommended. **Microbiological Effectiveness:** Bactericidal (incl. MRSA) in accordance to EN 1040, EN 1276, EN 13697; Levurocidal (Candida albicans) in accordance to VAH/DGHH, EN 1275, EN 1650, EN 13697; "restricted viruses" (effective against membrane covered viruses, i.e. HBV, HIV, HCV, H1N1, H5N1) and effective against Rota viruses. **Effectivity of saturated fluid:** Bacteria (incl. MRSA) in accordance to VAH/DGHH, EN 1040, EN 1276, EN 13697 (high bio load): Conc.: 1 min. Candida albicans in accordance to VAH/DGHH, EN 1275, EN 1650, EN 13697 (high bio load): Conc.: 1 min. Membrane covered viruses (i.e. HBV, HIV, HCV, H1N1, H5N1) in accordance to BGA(RKI)/DVV (with bio load): Conc.: 1 min. Rota-Viruses in accordance to BGW/DVV: Conc.: 1 min. **100 g saturated fluid contains:** 0,24 g Dialkyldimethylammoniumchlorid, 0,48 g Alkyl(Aryl)dimethylammoniumchlorid **Instruction for Disposing:** Packing material should be completely emptied of contents before disposal.

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Contents:  
Contenido/Contenut:  
Inhalt/Contenuto  
Inhoud: **100 pcs**

**DEHP**™ DE Healthcare Products  
DEDICATED TO EXCELLENCE

Distributed by: Distribuido por/Distribuit per/Vertriebt durch/Distribuito de:  
DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehpbond.com

# Safe R Non-alcoholic disinfection Wipes refill

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