**UK**
Instruction for use

## Universal Opaquer

For professional use only.

**INDICATIONS:**

Universal Opaquer is indicated for the coverage of highly discolored tooth structure or metal posts, pins or substructure.

**DESCRIPTION:**

Universal Opaquer is a light-activated, resin based, dental restorative material. Its low viscosity and precise syringe delivery make it easy to place.

**FORMULATION:**

Universal Opaquer is a composite based on a resin system of BIS-GMA and triethylene glycol dimethacrylate.

**INSTRUCTIONS FOR USE:**

- Prophy: The tooth and pins or posts should be cleaned by scaling and prophylaxis with flour of pumice.
- Isolation: Isolate the teeth. Use of a rubber dam is highly recommended.
- Pulp Protection: Base the preparation as needed. Calcium hydroxide cement may be used for covering small pulpal exposures. Any additional dentinal coverage desired can be achieved by using a glass ionomer lining cement. Eugenol containing cements must be avoided.
- Bonding: Use a state of the art dentin/enamel bonding system to bond this opaquer to the tooth structure. Modified bonding systems can be used to bond this opaquer to other kinds of materials used in restorative techniques. Cure the bonding resin before placing the opaquer. Follow the manufacturer’s instructions.
- Dispensing the Opaquer: Attach a syringe tip to the Universal Opaquer syringe. The opaquer can be applied directly to the tooth surface by exerting gentle pressure on the syringe plunger. Stop pressure on plunger before the desired amount of material is dispensed, since the flow of material from the syringe tip continues for a short time after the pressure is released. Slight back pressure on the plunger can help to decrease the continued flow of the resin out of the syringe.
- Placement and Curing: apply one thin coat (maximum ½ mm thick) to cover the discolored or metal area only. Cure with a dental curing light, for 40 seconds, with a wavelength around 470-480 nanometers and a minimum of 500 mW/cm² intensity.
- After curing, remove any excess with a sharp instrument.

**Warning:**

Dentists and assistants should wear gloves and protective eye wear. Patients also should wear eye protection.

Universal Opaquer contains methacrylate resins. Avoid use of this product on patients with known methacrylate allergies.

To reduce the risk of allergic response, avoid exposure to uncured resin.

- If contact with eyes occurs, flush with copious amounts of water.
- If contact with skin occurs, wash with soap and water.
- In case of ingestion, seek medical attention immediately.

Do not use eugenol-containing materials for pulp protection since they can retard the curing process.

Do not allow saliva or water contamination of etched tooth surface or unset material. Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.

- Test the dental curing light periodically to assure sufficient intensity of the light output.

**Storage:**

The system is designed to be stored between 50° - 75°F (10°-24°C).

Replace caps immediately after dispensing.

If stored in a refrigerator, bring to room temperature prior to use.

Do not store material in proximity to eugenol-containing materials.

Do not expose to elevated temperatures or intense light.

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**DE Healthcare Ltd**