SAFE R WIPES

DISINFECTANT WIPES (REFILL)

97-90887

Ready-to-use, wipes for the rapid disinfection of the surfaces of medical devices such as dental treatment units. Aldehyde-free.

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DEHP™ **DE Healthcare Products**
DEDICATED TO EXCELLENCE

Instructions for use: Place the refill bag completely in the dispenser container. Pull the wipe plucked from the middle of the roll and draw through the guide slot of the lid. Discard the first wipe and screw the lid tightly in place. Completely clean and disinfect the lid before next using. Wet surfaces and objects completely and allow solution to dry. Do not use on surfaces sensitive to alcohol (such as acrylic glass). Not suitable for the disinfection of invasive medical devices. Close container tightly after use. **Exposure time (applies to active substance solution):** • Bacteria/ fungi, MRSA, Tb - 1 min. [registered with VAH/DGHM [German Association for Applied Hygiene/ German Society for Hygiene and Microbiology]] • Rotaviruses - 30 sec. • Virucidal against enveloped viruses* (incl. HIV, HBV and HCV) - 30 sec. • Noroviruses - 60 sec. • Adenoviruses [type 2] - 2 min. • Polyoma SV viruses 40 - 15 min. *in accordance with RKI Recommendation, Bundesgesundheitsblatt [Federal Health Gazette] 08/2008 **Composition:** 1 container contains 150 ready-to-use wet wipes. 100 g of alcoholic solution contains 25 g of ethanol (94 %) and 35 g 1-propanol. Protect from frost, heat and sunlight. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of with household waste. Do not use near ignition sources. **Hazard notes and safety advice: DANGER: H226** Flammable liquid and vapour. **H318** Causes serious eye damage. **H336** May cause drowsiness or dizziness. **P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P280** Wear eye protection/ face protection. **P305 + P351 + P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P501** Dispose of contents/ container to an approved waste disposal plant.

Lot information only applies to the original filling.

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Lot information only applies to the original filling.

DE Healthcare Ltd
Unit 9, Kingshorpe Business Centre,
Studland Road, Kingshorpe,
Northampton NN2 6NE

Distributed by/Distribuido por/Distribuit per/Vertriebt durch/Distribuito da:
And in the USA/ Y en EEUU/ Et aux Etats-Unis/ Und in USA/ E in USA.

Denver, PA 17517 USA

Rev. 2014/02

15 °C
59 °F