KALZIUMHYDROXID

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Calcium Hydroxide • INSTRUCTIONS FOR USE

HIDROXIDO DE CALCIO • INSTRUCCIONES DE USO HYDROXYDE DE CALCIUM • MODE D'EMPLOI KALZIUMHYDROXYD • GEBRAUCHSANWEISUNG IDROSSIDO DI CALCIO • ISTRUZIONI PER L'USO

97-92275

CALCIUM HYDROXIDE

0) Composition

Calcium hydroxide (30%), barium sulphate (35%) and polypropylene glycol (35%).

1) Indications for use

This product is to be used only in a clinical or hospital environment, by qualified users. Users are responsible for checking whether this product can be used for the purpose intended.

Application fields:

- Temporary filling material for an infected root canal, especially in cases of exudation
- Partial pulpotomy
- Apexification

2) Contraindications

Do not use this product in case of known allergy to any of its components.

3) Warnings

This product can cause burns (calcium hydroxide). In case of eye contact, immediately wash out the eye thoroughly with plenty of water and consult a doctor.

Improper storage may alter the paste (drying of the paste or separation of its liquid and solid phases).

4) Precautions

- Do not fill past the apex
- Do not use in allergic patients
- Avoid direct exposure to sunlight
- Always cap the syringe after each use
- Store in a dry and cool area (10°-25°C).

Clean and disinfect the syringe after each use with a soft material impregnated with a detergent and disinfecting solution (bactericidal and fungicide solution containing no aldehydes).

5) Adverse reactions

Allergic reactions can be observed in patients who are sensitive to any component of this product. In such cases, stop using this product and remove it completely.

So far, no adverse effects have been reported to us.

6) Step by step instructions for use

- After preparing the root canal, clean and dry it
- Set up a disposable sterilised tip on the syringe and insert the provided ring for easier redirection of the tip
- Insert the tip into the root canal down to the working length, slightly remove it and begin to inject the product
- Fill the canal slowly from apex to pulp chamber by pushing the syringe plunger gently.
- Wipe away any paste excess with a sterilized cotton pellet
- Remove and eliminate the used tip, fix a new one and cap it.

7) Disinfection, cleaning and sterilisation

The disposable tips are to be autoclaved at 134° for 18 minutes prior to use.

The tips are single-use and should be disposed of after use.

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