Safe R Surface

IMPROVED
FORMULA

97-92540

Surface Disinfection

- Ready-to-use rapid surface disinfectant for non-invasive medical device.
- Desinfectante rápido, listo para usar, para el área médica y dental farmacéutica no invasiva.
- Désinfectant rapide prêt à l'emploi les petites surfaces médicaux-dentaires non invasif.
- Gebrauchsfertige Schnelldesinfektion für den ärztlichen und zahnärztlichen Bereich zur Desinfektion von nichtinvasiven Medizinprodukten.
- Disinfettante pronto per l'uso per le applicazioni mediche e odontoiatriche farmacéutica non invasiva.
- Snel werkende gebruiksklare desinfectans voor het medisch en tandheelkundig gebruik van niet-invasieve medische producten.

Indications for use: Spray the respective area from a distance of 30 cm until completely covered. Leave to rest. Wipe residue with an uncoloured disposable cloth. Only to be used on alcohol compatible materials (i.e. not suitable for acrylic glass). **Microbiological Effectiveness:** Bactericidal (incl. MRSA) according to VAH/DGHM, EN 1040, EN 1276; levurocidal (Candida albicans) according to VAH/DGHM, EN 1275, EN 1650; tuberculocidal according to VAH/DGHM, EN 14348; "restriected virucidal" (effective against membrane covered viruses, i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV, virucidal against Adeno-viruses according to EN 14476. **Effectivity:** Bactericidal (incl. MRSA) in accordance to VAH/DGHM, EN 1276 (high bio load); Conc.: 1min. levurocidal (Candida albicans) in accordance to VAH/DGHM, EN 1650 (high bio load), EN 1275 Conc.: 1min. TBb (Tuberculosis) inactivation in accordance to VAH/DGHM, EN 14348 (high bio load); Conc.: 1min. Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA(RKI)/DVV with bio load; Conc.: 30sec. Adeno-Viruses (EN 14476); Conc.: 30sec. **100g contains:** 43,0g Ethanol, 8,0g 1-Propanol, 0,04g Didecyl/dimethylammoniumchlorid. **Safety Instruction:** R 10 Flammable. S 7 Keep container tightly closed. S 16 Keep away from sources of ignition. No smoking. S 60 This material and its container must be disposed of as hazardous waste. Flammable! Do not spray into flame! Flashpoint according to DIN 51755: approx. 25°C. Max. quantity 50ml per m². The total quantity per area is not allowed to be more than 100ml per m² of the ground area. Keep container tightly closed. Packing material should be completely emptied of contents before disposing.

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Contents:
Contenido:
Inhalt:
Contenuto:
Inhoud:

5L e

DEHP™ DE Healthcare Products
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribué par/Vertrieb durch/Distribuito da:
DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehprand.com

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DEHP™ **DE Healthcare Products**
DEDICATED TO EXCELLENCE

Distributed by/Distribuido por/Distribué par/Vertriebt durch/Distribuito da:
DE Healthcare Products, Gillingham ME8 0SB U.K., www.dehprand.com