User Manual

Colourless PMMA discs are mainly suitable for the manufacturing of splints and other oral cavity devices. They can also be used for the manufacturing of trial frameworks destined to be trialed within the oral cavity, for working models and as a base of the manufacturing of precision casting impressions for metal or ceramics. The braces are first designed with through the appropriate CAD. When planning the frameworks for precision casting, the thickness and cross-sections that apply to the final material need to be taken into consideration. For milling the, for acrylic, appropriate tools should be used. For polishing please use pumice stones, pastes and soft polishing brushes. It is critical that the material does not overheat!

Colourless PMMA discs are made from pure polymethyl methacrylate.

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Zirlux Splint Transparent

9793712

Zirlux Splint Transparent,
diameter 98,5; thickness 14 mm
with shoulder

9793713

Zirlux Splint Transparent,
diameter 98,5; thickness 16 mm
with shoulder

9793714

Zirlux Splint Transparent,
diameter 98,5; thickness 18 mm
with shoulder

9793715

Zirlux Splint Transparent,
diameter 98,5; thickness 20 mm
with shoulder

9793716

Zirlux Splint Transparent,
diameter 98,5; thickness 25 mm
with shoulder

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 **DE Healthcare Ltd**
(manufacturer)
**Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.**

Distributed by/Distribuido por/Distribué par/Vertrieb durch/Distribuito da:
And in the USA/ Y en EEUU/ Et aux Etats-Unis/ Und in USA/ E in USA:

 **0473**
**41 Weaver Road
Denver, PA 17517 USA**

Rev. 2015/02