TEMPORARY CROWN AND BRIDGE RESIN

Instructions for use

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liquid / líquido / liquide / Flüssigkeit / liquido: 97-94466

Direction for use

The temporary crown and bridge resin is a powder / liquid system for fabricating temporary crowns and bridges. It is a cold-cured polymer with a polymethyl methacrylate base that contains just 0.8% residual monomer.

The temporary crown and bridge resin is suitable for both the direct and indirect methods.

Mixing and Adjusting the Colour

Mix the temporary crown and bridge resin in a cup.

Suggested mixing ratio is 2 parts powder to 1 part liquid. A creamy dough like viscosity is 1.8-2.2g powder to 1g liquid. Fill the cup with liquid, for an average crown 8 drops of liquid and 0.3g or powder is sufficient. Colour can be adjusted by mixing light and dark powders together.

Indirect Method

Insulate the diagnostic model with silicone spray. Use a deep-drawing unit to coat a matrix foil. Cut off excess margins. If bonding to the temporary appliance is not required, use a deep drawing film made from polypropylene. If bonding to the temporary appliance is desired to stabilise slightly tapering margins, we recommend the use of commercial deep-drawing film (0.5 mm) that bonds with acrylate. Mix the temporary crown and bridge resin as described and pour directly into the matrix. Do not start application more than 2 minutes after adding the powder to the liquid. The matrix foil must be positioned exactly on the model in order to prevent bite increase and thus later corrections in the mouth.

After approximately 6 minutes, carefully lift the temporary appliance with matrix off the model. The 1-2 minute elastic phase will begin. The overlapping margins can now be removed with scissors. The temporary appliance remains on the model for final curing (about 2 minutes). After final curing, grind and polish the margins. The temporary crown and bridge resin is repairable.

Direct Method

Impression Technique

Before preparing the tooth, take an alginate or silicone impression and store it in the usual way. After preparation, mix the temporary crown and bridge resin as described above. Remove interdental septa if necessary and blow the impression dry. Pour the mixed temporary crown and bridge resin into the desired area of the impression. Setting time is 2 minutes. Before reinserting the impression into the mouth ensure the prepared stumps are coated with Vaseline. After approximately 3 ½ minutes the temporary appliance can be removed during the rubber elastic phase. This will prevent the resin being accidentally bonded to the prepared teeth. If the appliance does adhere to the preparation it must be removed carefully and returned to the impression for curing. Once cured the appliance can now be trimmed.

Matrix Technique

As already described, a foil that is 0.6 - 0.8 mm thick is deep-drawn over a diagnostic model insulated with silicone spray. Mix the temporary crown and bridge resin as described above and cast it in the matrix. Once the powder is added to the liquid it must be used within 2 minutes. The matrix foil must be placed accurately on the teeth. The patient should close their mouth to hold in position. Carefully remove the appliance with matrix after 3 ½ minutes. And follow processing as above if necessary.

Storage

The material should be stored at 10 - 25°C. Seal the container securely after use. Keep away from ignition sources and do not smoke.

Troubleshooting

Problem	Cause	Solution
Plastic not curing properly	wrong mixing ratio	- follow directions for use - use correct mixing ratio
Plastic is curing too quickly /too slowly	Unfavourable ambient temperature	Adjust the procedure High temperatures accelerate polymerisation, while low temperatures slow it down
Colour issue	Light and dark powders can be mixed individually	Select another powder mixing ratio.
Temporary appliance is adhering to the prepared tooth	It has been left too long in the mouth	Follow the directions for use

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TEMPORARY CROWN AND BRIDGE RESIN LIQUID + POWDER

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Danger. Liquid contains methyl methacrylate.

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/ open flames/hot surfaces. - No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/shower. Store locked up. Dispose of contents/container in accordance with local/regional/national/international regulations.

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**DE Healthcare Ltd**